

CLEANING

1. Clean hands effectively and dry before handling mask.
2. Hold the mask with the inside facing up. With the disinfectant spray nozzle pointing at the mask, hold the bottle perpendicular 10cm away from the mask, press the nozzle all the way down and release it five times whilst maintaining the distance from the mask at all times. *WARNING: Spraying in one position, too close to the mask or spraying multiple times will lead to over-wetting that may cause the mask to become ineffective.*
3. Turn over the mask and place on a clean, dry surface. Repeat spraying on outside of mask as described in Step 2.
4. Leave mask to air dry naturally for 1 hour at room temperature before storing away.

If the mask was not sprayed satisfactorily, wait for 1 hour before repeating the cleaning process.



DISCLAIMER

As with all protective face masks and Personal Protective Equipment (PPE) in general, use of this product does not guarantee that the user will not become exposed to any airborne bacteria and/or viral infections. However, the risk of such exposure will be significantly reduced by the use of this product in accordance with the instructions for use included in this document.